

# ARGEN<sup>®</sup> RESINS

## CURE BOX COMPATIBILITY

| Unit                                      | Model           | ARGEN MODEL PRO                  | ARGEN MODEL   | ARGEN FLEX  | ARGEN GUIDE   | ARGEN TRAY  |
|---|-----------------|----------------------------------|---|---|---|---|
| OtoFlash                                  | G171            | 250 flashes per side             | 2000 flashes per side   | 2000 flashes per side   | 2000 flashes per side   | 2000 flashes per side   |
| Uvitron                                   | Intelliray 400W | Not available                    | 2 minutes per side  | 2 minutes per side  | 150 seconds per side at 50% intensity                                     | 2 minutes per side  |
| Photopol (with Asiga <sup>®</sup> )       | A5408V          | Not available                    | 10 minutes total  | 50 minutes total  | 40 minutes total  | 50 minutes total  |
| NextDent (3D Systems)                     | LC-3DPrint Box  | Not available                    | Not available   | 15 minutes total  | 30 minutes total  | 15 minutes total  |
| SprintRay                                 | Pro Cure        | Not available                    | 15 minutes per side @ 50C   | 30 minutes total @ 50C  | 45 minutes total @ 50C  | 15 minutes total @ 50C  |
| Rapid Shape                               | RS Cure         | Not available                    | 5 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum                         | 6 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum | 3 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum | 6 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum |
| Phrozen                                   | Cure            | Not available                    | 20 minutes per side   | 40 minutes per side   | 20 minutes per side   | 30 minutes per side   |
| SHINING 3D                                | Fab Cure        | Not available                    | Pending   | 20 minutes per side   | 20 minutes total  | 40 minutes total  |
| Eurolight (with Flashforge Hunter)        | N/A             | Not available                    | Pending   | Not available   | 3.5 minutes per side  | Pending   |
| MeccatroniCore (with Asiga <sup>®</sup> ) | BB Cure         | Not available                    | 10 minutes total  | Not available   | Not available   | 50 minutes total  |
| Wicked (with Asiga <sup>®</sup> )         | CureBox         | Not available                    | Argen MODEL Beige: 20 minutes per side @ 60C<br>Argen MODEL Light Grey: 40 minutes per side @ 60C | 20 minutes per side @ 60C   | 5 minutes per side @ 60C  | 5 minutes per side @ 60C  |
| Ackuretta (with Ackuretta)                | Curie           | 5 minutes total, P-16 D-10 BL-ON | 10 minutes total, P-16 D-10 BL-ON   | 10 minutes total, P-16 D-10 BL-ON   | 5 minutes total, P-16 D-10 BL-ON  | 10 minutes total, P-16 D-10 BL-ON   |
| Ackuretta (with Ackuretta)                | Curie Plus      | 5 minutes total, P-16 D-10 BL-ON | 10 minutes total, P-16 D-10 BL-ON   | 10 minutes total, P-16 D-10 BL-ON   | 5 minutes total, P-16 D-10 BL-ON  | 10 minutes total, P-16 D-10 BL-ON   |

\*Not a trademark of The Argen Corporation.

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126  
argen.com | customersuccess@argen.com | (800) 255-5524



BD-18938-Ver3-04/24