# Programat<sup>®</sup> EP 3010<sub>G2</sub>

|                                  |    |
|----------------------------------|----|
| Short Instructions               | 2  |
| Kurzanleitung                    | 13 |
| Mode d'emploi version simplifiée | 24 |
| Istruzioni d'uso brevi           | 35 |
| Instrucciones breves             | 46 |
| Guia Rápido                      | 57 |

# Programat® EP 3010<sub>G2</sub>

## Short Instructions

### Dear Customer

Thank you for having purchased the Programat® EP 3010 G2. In order to render the setting-up and the initial use of the furnace as easy as possible, we have prepared these short instructions.

**These short instructions replace by no means the detailed Operating Instructions.  
We therefore kindly ask you to thoroughly read the Operating Instructions and follow the notes contained therein.  
You can find the Operating Instructions in the download section of <http://www.ivoclarvivadent.com>.**

Enjoy working with the Programat EP 3010 G2.

# 1. Intended use

## Intended purpose and use

The Programat EP 3010 G2 is a ceramic furnace for the dental field. It was specifically developed for use in dental laboratories. The firing chamber may be heated up to max. 1200 °C (2192 °F) by means of a heating element. Furthermore, the firing chamber has been designed in such a way that a vacuum may be created with a vacuum pump. Electronic components with the corresponding software monitor and control the firing programs. Additionally, the set and actual temperatures are continuously compared. Use the Programat EP 3010 G2 exclusively for the firing process of dental ceramic materials. Other uses than the ones stipulated, such as the cooking of food, firing of other materials, etc., are contraindicated. The manufacturer does not assume any liability for damage resulting from misuse. The user is solely responsible for any risk resulting from failure to observe these Instructions.

Further instructions to assure proper use of the furnace:

- The instructions, regulations and notes in these Short Instructions must be observed.
- The instructions, regulations and notes in the material's Instructions for Use must be observed.
- The furnace must be operated under the indicated environmental and operating conditions (see Chapter 6).
- The Programat EP 3010 G2 must be properly maintained.

## Additional information

The device has been developed solely for use in dentistry. Start-up and operation should be carried out strictly according to the Operating Instructions. Liability cannot be accepted for damages resulting from misuse or failure to observe the Instructions. The user is solely responsible for testing the apparatus for its suitability for any purpose not explicitly stated in the Instructions.

Information on cleaning the furnace can be found in the Operating Instructions. Information about the correct positioning of materials with regard to this furnace can be found in the Operating Instructions of the furnace or the Instructions for Use of the respective material.

## Safety notes

| Hazardous area                                 | Type of risk             |
|--|--------------------------|
| Firing chamber                                 | Burn hazard              |
| Opening/closing mechanism                      | Risk of crushing         |
| Electrical components (cables and connections) | Risk of electrical shock |

## Safety information for the operation

The device may only be operated by authorized and trained technical personnel.

- Keep unauthorized persons, such as patients, children and animals, away from the device.
- In all cases of doubt regarding the safety of the device, switch off the device and take suitable measures to prevent further use.
- Prior to connecting power or operation, check the device, the accessories and protective equipment for any damage.
- Do not use damaged, non-functioning equipment or accessories. Inform your authorized service partner.
- In order to ensure the product reliability and warranty services, the device must be exclusively operated with the accessories from Ivoclar Vivadent, in particular the original power adapter.
- The user bears the risk when using non-approved accessories.
- To prevent damage to the device and a reduction of the device performance, observe the cleaning requirements and cycles.
- Only operate the device unsupervised if the operating conditions for unsupervised operation described below are fulfilled.

## Unsupervised operation

The device may be operated unsupervised, provided the national and local laws and provisions allow for such action and provided that they are observed. Furthermore, the requirements of the respective insurance company must be met.

- Never use the device if the work environment is heavily soiled.
- Protect the device against unauthorized access.

## Safety notes

This furnace has been designed according to EN 61010-1 and has been shipped from the manufacturer in excellent condition as far as safety regulations are concerned. To maintain this condition and to ensure risk-free operation, the user must observe the notes and warnings contained in these Operating Instructions.

- The user must especially become familiar with the warnings and operating conditions to prevent injury to personnel or damage to materials. The manufacturer is not responsible for damage resulting from misuse or failure to observe the Operating Instructions. Warranty claims cannot be accepted in such cases.
- Before switching on the furnace, make sure that the voltage indicated on the rating plate complies with your local power supply.
- The mains socket must be equipped with a residual current operated device (FI).
- The power plug acts as a circuit breaker and may only be connected with an easy-to-access power socket with protective contact.
- Use only the supplied original power cord.
- Place the furnace on a fire-proof table: Observe local regulations, (e.g. distance to combustible substances or objects, etc.).
- Make sure that the air vents on the sides and the rear are always unobstructed.
- Do not touch any parts that become hot during operation of the furnace. Burn hazard!
- When removing hot components from the firing chamber (e.g. firing table, firing tray), make sure to place them on a fire-proof surface.
- Clean the furnace only with a dry, soft cloth. Do not use any solvents!  
Disconnect power before cleaning and allow the furnace to cool down!
- The furnace must be cool before it is packed for transportation.
- Use original packaging for transportation purposes.
- Before calibration, maintenance, repair or change of parts, the power must be disconnected and the furnace must be cool if it needs to be opened.
- If calibration, maintenance or repair has to be carried out with the power connected and furnace open, only qualified personnel who are familiar with the risks and dangers may perform the procedures.
- After maintenance, the required safety tests (high voltage resistance, protective conductor, etc.) have to be carried out.
- Make sure that only fuses of the indicated type and rated current are used.
- Use only original spare parts.
- If it is assumed that safe operation is no longer possible, the power must be disconnected to avoid accidental operation.
- If the furnace is visibly damaged, the power must be disconnected to avoid accidental operation.
- Check the correct function of the device after storage under unfavourable conditions over an extended period of time.
- If the furnace does not work properly, the power must be disconnected to avoid accidental operation.
- The temperature range for faultless operation is +5 °C to +40 °C (41 °F to 104 °F).
- If the furnace has been stored at very low temperatures or high atmospheric humidity, the furnace has to be left to adjust to room temperature for approx. 4 hours. Do not connect the power yet.
- The furnace is tested for use at altitudes of up to 2000 m above sea level.
- The furnace may only be used indoors.
- Before leaving the factory, the furnace functions were tested for several hours. It is therefore possible that these tests have caused slight discolouration of the insulation. Nevertheless, your Programat is still a brand new furnace.

Any disruption of the protective conductor either inside or outside the furnace or any loosening of the protective conductor may lead to danger for the user in case of malfunction. Deliberate interruptions are not tolerated.

Materials developing harmful gases must not be fired!

**Warnings regarding the dismantling of the heating muffle**

This product contains ceramic fibres and may release fibre dust. Fibre dust has proved to be carcinogenic in animal experiments. The heating muffle may only be dismantled by a qualified After Sales Service Centre. Information regarding the Safety Data Sheet is also available from your After Sales Service Centre.

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: [www.ivoclarvivadent.com](http://www.ivoclarvivadent.com), and your responsible competent authority.
- The current Operating Instructions are available in the download section of the Ivoclar Vivadent AG website ([www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)).

## Explanation of the symbols:

| Symbol | Note                                    |
|--------|---|
|        | Burn hazard                             |
|        | Risk of crushing                        |
|        | The Operating Instructions must be read |

## 2. Installation

Remove the furnace components from their packaging and place the unit on a suitable table. Please observe the instructions on the outer packaging. Make sure the voltage indicated on the rating plate complies with the local power supply. If this is not the case, the furnace must not be connected.

### A Mounting the cooling tray

Remove both screws for the cooling tray (1).

### B Place the cooling tray (2) on the frame plate (3).

Make sure that the cooling tray is correctly positioned on the frame plate.

### C Secure the cooling tray with the two screws (1).


### D Placing the firing plate

The firing plate (4) can now be placed on the firing plate holder (5).

### E Mounting the furnace head

The furnace head is best mounted with the rear panel of the furnace pointing towards the user. Lift the furnace head with both hands as shown in the picture and place it on the furnace head mounting.

- F  Mount the furnace head as shown in the picture until it audibly snaps into place. Make sure that the firing plate or the insulation is not damaged by mounting the furnace head.
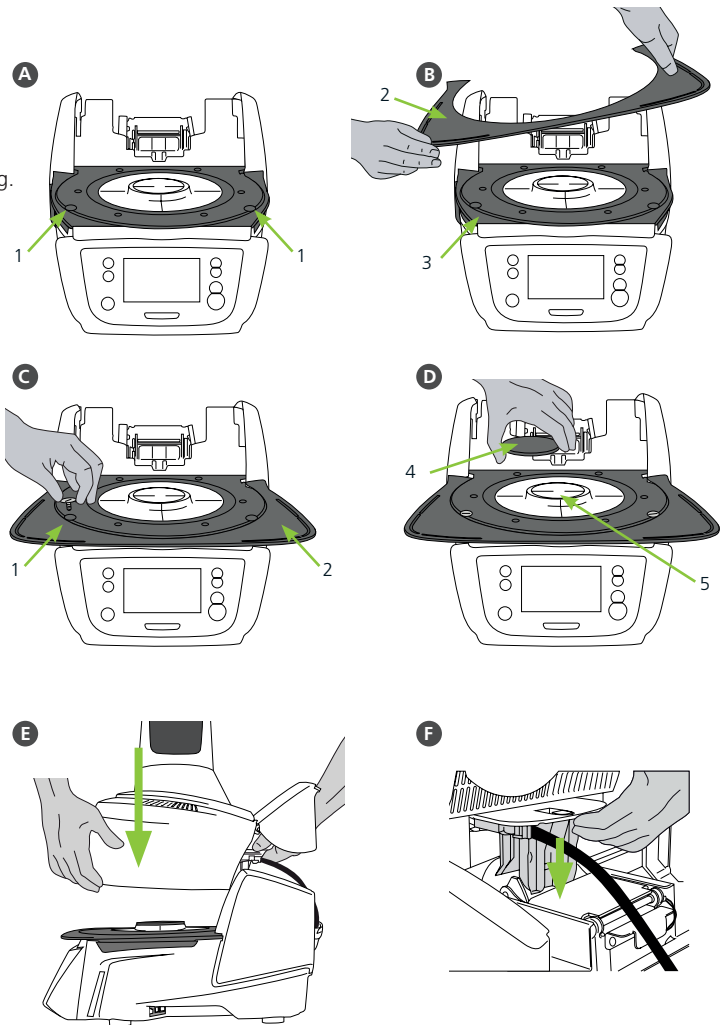


**G Establishing the connections**

Connect the cables of the furnace head with the furnace base.

Proceed as follows:


- Connect the vacuum hose (6).
- Insert the thermocouple plug (7). (Make sure that the polarity of the plug is correct.)
- Insert the press drive plug (8).
- Insert the heater plug (9).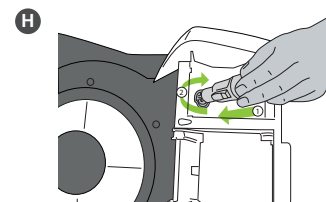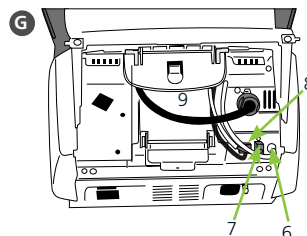


- H** Insert the heater plug into the intended connector. Secure the heater plug by rotating it 45°.

**I Closing the covering hood**

Once all cables are correctly connected to the furnace base, close the cover (10) to protect the connections until you feel it click into place.

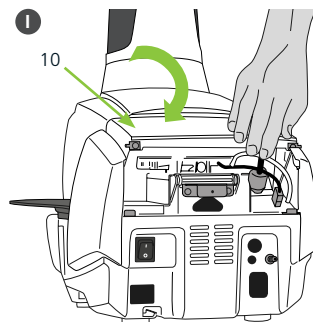**J Mounting the connection cover**

The connection cover (11) can now be mounted and secured with the screws.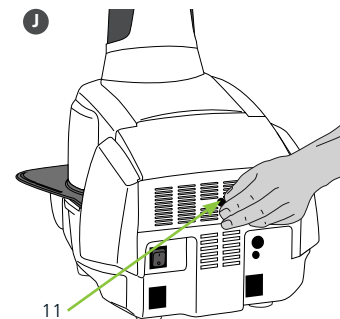


The furnace may only be used with the covering hood closed and the connection cover mounted!

**K Establishing additional connections****Power connection**

Make sure that the voltage indicated on the rating plate complies with the local power supply. Subsequently, connect the power cord (12) with the power socket of the Programat (13).


**Connecting the vacuum pump**

Connect the vacuum pump plug (14) with the vacuum pump socket (15). We recommend using only a vacuum pump from Ivoclair Vivadent, since these pumps are especially coordinated with the furnace. If other pumps are used, please observe and do not exceed the indicated maximum power consumption.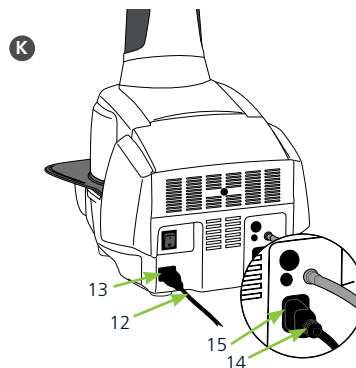

### 3. User interface

The Programat EP 3010 G2 has a wide-screen colour display. The keypad and the touch screen allow user-friendly handling. By tapping slightly with your fingertips on the touch screen, the buttons are activated and the furnace performs the requested function.

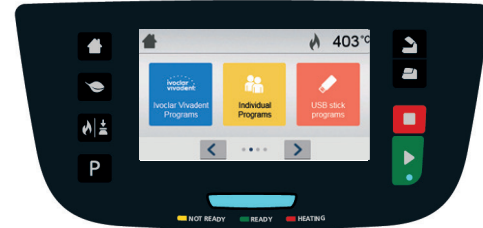#### Keypad:

##### Home

Returning to Home screen (main menu)

##### Program key

**Pressing once:** The currently selected program is displayed.

**Pressing twice:** The program selection via number entry is accessed.

##### Change operating mode key

By pressing this key, you can switch between the firing and the press mode.

##### Power-saving key

This key activates the power-saving function.

##### Open furnace head key

##### Close furnace head key

##### START key (Start LED)

Starts the selected program. The green LED indicates that a program is running. If the program is paused (1 x STOP), the Start LED flashes until renewed pressing of START results in the program being resumed.

##### STOP key


A running program can be paused by pressing the STOP key and terminated by pressing the key twice. Movement of the furnace head can be stopped at any time by pressing STOP. Acoustic signals can be turned off by pressing the STOP key.

## 4. Operation

### Start-up

Connect the power cord with the wall socket and put the On/Off switch at the rear of the furnace to position "I". After the self-test is completed, the furnace will automatically display the screen to select the desired operating mode. Select the firing or press mode.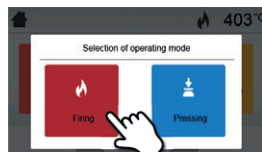

### 4.1 Operation in the FIRING mode

#### Step 1:

##### Select the program type.

The furnace offers several types of programs:

- firing programs for Ivoclar Vivadent materials
- 300 free, individually adjustable firing programs

#### Step 2:

##### Select the program group.

The program types are subdivided into program groups. Each program group consists of 20 programs.

#### Step 3:

##### Select the desired firing program.


In this screen, firing programs can be started or edited.


#### Step 4:

##### Edit the firing program.

To edit a parameter, press the respective button and enter the desired value. Subsequently, confirm the entry by pressing the green button.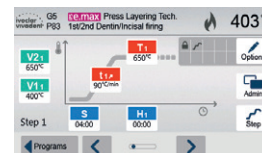

When the furnace is delivered ex factory, Ivoclar Vivadent programs are write-protected. The write-protection can be deactivated in the Options menu, where additional program options are available.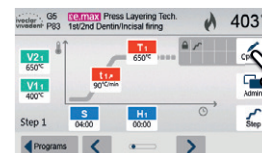

#### Step 5:

##### Load the furnace and start the program.

Open the furnace head using the OPEN FURNACE HEAD key and position the firing tray with the object in the furnace. Activate the program by pressing the START key.

## 5. Editing the firing parameters

### Operation in the PRESS mode

#### Step 1:

##### Select the program type.


The furnace offers several types of programs:

- press programs for Ivoclar Vivadent materials
- FPF Fully automatic press function for IPS e.max Press materials
- 20 free, individually adjustable press programs

#### Step 2:


##### Select the desired press program.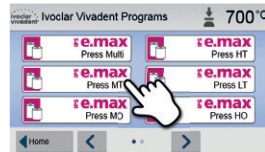

#### Step 3:

##### Select the investment ring size.

Select the desired investment ring size (100 g / 200 g / 300 g). In the Ivoclar Vivadent press programs, not all investment ring sizes are available for each material.

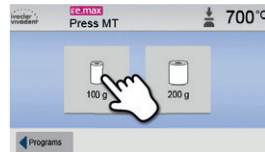#### Step 4:

##### Load the furnace and start the program.

Load the furnace with the preheated investment ring and activate the program by pressing the START key.

### Editing press programs

The Ivoclar Vivadent press programs are equipped ex factory with the optimum parameters for Ivoclar Vivadent pressed ceramic materials. These programs are write-protected and cannot be edited. Individual press programs can be edited in the same way as the firing programs (see 4.1).

**Please see the Operating Instructions for more detailed information.**

## 6. Technical data and operating conditions

### 6.1 Technical data

|  |   |
|--|---|
| Power supply   | 110–120 V / 50–60 Hz<br>200–240 V / 50–60 Hz  |
| Overvoltage category   | II  |
| Contamination level  | 2   |
| Acceptable voltage fluctuations  | ± 10 %  |
| Max. power consumption   | 12 A at 110–120 V<br>8 A at 200–240 V   |
| Acceptable data for pumps of other manufacturers:<br>Max. output:<br>Final vacuum: | 250 W / max. leakage current 0.75 mA<br>< 50 mbar<br>Only tested pumps should be used |
| Electrical fuses   | 110 – 120 V:<br>125 V / T15A (heating circuit)<br>125 V / T5A (vacuum pump)           |
|  | 200 – 240 V:<br>250 V / T8A (heating circuit)<br>250 V / T3.15A (vacuum pump)         |
| Dimensions of electrical fuses   | 110–120 V: Diameter 6.3 x 32 mm<br>200–240 V: Diameter 5 x 20 mm                      |
| Dimensions of the closed furnace   | Depth: 465 mm<br>Width: 320 mm / 390 mm (with cooling tray)<br>Height: 550 mm         |
| Usable size of the firing chamber  | Diameter: 90 mm<br>Height: 80 mm  |
| Max. firing temperature  | 1200 °C (2192 °F)   |
| Weight   | 18.3 kg   |

### Safety notes

The furnace complies with the following standards:

IEC 61010-1:2010  
IEC 61010-1:2010/AMD1:2016  
EN 61010-1:2010/A1:2019  
UL 61010-1:2012/R:2019-07  
CAN/CSA-C22.2 No. 61010-1:2012/A1:2018-11

EN 61010-2-010:2020  
IEC 61010-2-010:2019  
UL 61010-2-010:2019  
CAN/CSA-C22.2 No. 61010-2-010:2019

Radio protection, electromagnetic compatibility: EMC tested

### 6.2 Acceptable operating conditions

- Acceptable ambient temperature range: +5 °C to +40 °C (41 °F to 104 °F)
- Acceptable humidity range: Relative humidity 80% for temperatures up to 31 °C (87.8 °F) gradually decreasing to 50% relative humidity at 40 °C (104 °F), condensation excluded.
- Acceptable ambient pressure: The furnace is tested for use at altitudes of up to 2000 m above sea level.

### 6.3 Acceptable transportation and storage conditions

- Acceptable temperature range: –20 °C to +65 °C (–4 °F to 149 °F)
- Acceptable humidity range: Max. 80% relative humidity
- Acceptable ambient pressure: 500 mbar to 1060 mbar

Use only the original packaging together with the corresponding foam material for shipping purposes.

## 7. Maintenance, cleaning, diagnosis

Chapter 7 of the comprehensive Operating Instructions for this device contains detailed descriptions of the maintenance, cleaning and diagnostic procedures for the device.

Only the tasks listed may be carried out by qualified dental technicians or dental laboratory professionals. All other tasks must be performed by qualified service personnel at a certified Service Centre.

### General information on the control and maintenance tasks

The time for these maintenance procedures depends on the frequency of use and the working habits of the users. For that reason, the recommended times are only approximations.

## 8. Disposal of the device

Chapter 2 of the comprehensive Operating Instructions for this device contains information about the proper and correct disposal of the device.

The device must not be disposed of in the normal domestic waste. Please correctly dispose of old furnaces according to the corresponding EU council directive. Information regarding disposal may also be found on the respective national Ivoclar Vivadent website.

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## Ivoclar Vivadent Limited
## Ivoclar Vivadent, Inc.