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Regolamento UE2016/425

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Regulation requirements

Regulation UE2016/425

requirements for designing and manufacturing PPE;

UNI EN 166:2004 - Personal protection of the eyes

Specifications, CE 765/2008 - Graphic signs - Colour and safety signs

Recorded safety signs.

Istruzioni d'uso

ATTENTION/WARNING: put the PPE on before exposing yourself to risks

Markings:

Markings on the lenses: L 1 3 S CE - 1 (OPTICAL CLASS), 3 (FIELD OF USE),
S (INCREASED ROBUSTNESS)

Markings on the frame: L EN 166 3 S CE - S (INCREASED ROBUSTNESS),
EN 166 (PRESENT LEGISLATION)

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)

16030 Tribogna (Genova)

Tel. +39 0185 939481 - Fax. +39 0185 938595

info@larident.it - www.larident.it

PROTECTIVE VISOR R62 INSTRUCTIONS

The protective visor is classified as PPE class II, it protects the face from drops/sprays of water and saliva and reduces risks of infection.

Very comfortable with characteristics that mean it can be worn safely and without any annoyance for hours.

The practical metal clips, that keep the shield attached to the frame, make assembling particularly easy.

Completely transparent polycarbonate protective shield that can be replaced easily and quickly.

Larident Srl

Via Lamaneigra 12R (Ponte Rimassa)

16030 Tribogna (Genova)

Tel. +39 0185 939481 - Fax. +39 0185 938595

info@larident.it - www.larident.it


PROTECTIVE VISOR R62 ASSEMBLING

1
Remove the protective film from both sides

2
The shield must be positioned on the outside of the frame, with the nasal opening facing downwards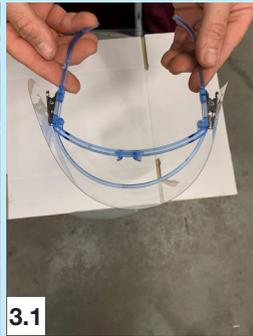

3
Insert the shield into clips at the sides of the frame, clicking it into place

4-5
4
Remember to adjust the visor to the shape of the face so it's well adherent to the cheekbones

5
The visor is ready

WARNINGS: IF THE VISOR IS SCRATCHED OR MISTED UP IT MUST BE REPLACED WITH A NEW ONE.

PROTECTIVE VISOR R62 DESCRIPTION

Warnings

The materials used to make our visors can cause allergies to very sensitive skins. Pay attention and handle with care.

Maintenance

Do not use alcohol solutions, they can cause scratches and misting up.

CLEANING

For the shield and the frame use disinfectant, no alcohol solutions based on soaps or other suitable detergents. After cleaning rinse away all residue detergent under running water.

HOW TO USE

- Use for protecting the face from drops/sprays of water and saliva.
- Must be used by one person.
- It can be used with other PPE, like FFP masks or reading glasses.

INSTRUCTIONS: HOW TO WEAR

Put the PPE on like reading glasses, checking that the shield fully covers all of the user's face and that the higher part of the visor is adherent to the forehead. For a correct positioning of the visor check that the nose piece is well adherent, adjusting the head strap to the size of the user's head. Once verified that the visual field is good and not blurred the PPE is positioned/worn correctly.

ADVICE

- Accurately examine the PPE before every use, and do not use it if there are obvious signs of it having been previously used or damaged
- Do not tamper with the PPE
- Follow the instructions for assembly indicated in the previous paragraph
- if the PPE shield is damaged or scratched it must be replaced