FRANÇAIS

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ENGLISH

INDICATIONS: NOBIL GRIP is an anaerobic glue for the production of dental prostheses to be used in threaded parts of attachments and anchors, screws, prosthetic implants and orthodontics expansion screws.

NOBIL GRIP assures the screwing in mouth over the time, by maintaining however the possibility of unscrewing and interchangeability.

NOBIL GRIP is non-toxic and long lasting in mouth.

APPLICATION: clean and dry perfectly the parts to be bonded.

Immerse the thread in a small quantity of NOBIL GRIP then screw the parts and remove very well the excess material.

The prosthesis can be applied immediately after.

The maximum curing is achieved after three hours about.

NOTE: perform the bonding operations always outside of the oral cavity. Remove well the excess material by flushing under running water.

SAFETY INFORMATION:

Warning

H319 Causes serious eye irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

P261 Avoid breathing vapours.

P280 Wear protective gloves.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

P337+P313 If eye irritation persists: Get medical advice/attention.

P501 Dispose of waste and residues in accordance with local requirements

Contains:

Hydroxypropyl methacrylate, 2,2-Ethylene-dioxydiethyl dimethacrylate, Methacryloyloxyethyl succinate, cumene hydroperoxide, methacrylic acid, 2-hydroxyethyl methacrylate, 1-Acetyl-2 Phenylhydrazine

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