CAD
SYSTEM

LASCOD

oklurest

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APPLICATIONS

Oklurest is an addition silicone specifically intended for orthodontic occlusion registration, registration keys for gnathological registrations, inter-maxillary registration keys for centered positions, eruptions and ectopic eruptions, registration for cephalometric analysis which also require subsequent scanning with CAD systems.

CHARACTERISTICS

- Scannable using optical/laser/tactile scanning systems
- Easy and safe handling thanks to the 1:1 double-cartridge system
- Maximum control after positioning
- Thixotropic
- Viscosity imperceptible for the patient
- High details definition
- Fast setting time
- Easy repositioning in the mouth and on the models after hardening
- Can be finished using burs

METHOD FOR USE

Load the cartridge into the dispenser (1) and after removing the cap, extrude a small amount of silicone from the two nozzles, moving the lever several times (2). Insert the mixing tip into the cartridge. Actuate the lever forcefully so as to extrude the silicone from the mixing tip. Apply Oklurest directly and without interruption onto the occlusal surface of the teeth (3). The operator then asks the patient to bite down with maximum intercuspation (4); since it is imperceptible, Oklurest facilitates the closure in a natural manner. Wait for complete hardening and remove the obtained impression.

The thixotropic consistency of Oklurest facilitates its control and thus prevents the material from entering the undercut areas and from dripping during positioning on the dental arches independently of the method used (occlusal or vestibular).

TECHNICAL DATA

Base paste/catalyser ratio	1:1
Working time (23°C/73°F)	45"
Minimum time in mouth	60"
Detail reproduction	5µm
Linear dimensional change at 24 hours	-0.02%
Hardness Shore A	95

WARNINGS/PRECAUTIONS

The times indicated above may be influenced by the following factors:

- Changes to the temperature of the material and/or of the environment (higher temperatures reduce the times, lower temperatures increase them).
- Humidity.

Avoid direct or indirect contact with latex rubber dams, gloves or with components of such material. Contact with latex inhibits the hardening of polyvinylsiloxanes. Accurately wash your hands and use vinyl/nitrile gloves.

Rinse the impression with plenty of water and disinfect it immediately.

Store the impression at room temperature without particular precautions other than protection from mechanical stress.

For correct storage of the remaining product in the cartridge leave the mixing tip installed until the next use. Before installing a new mixing tip, make sure that the silicone properly flows from the nozzles. If the cartridge remains unused for a long period, it may become blocked by hardened silicone, which must be removed with a thin instrument before a new mixing tip is placed on.

OTHER AVAILABLE SILICONES

For dental clinics: SILAXIL condensation silicone; GHENESYL addition silicone.

For laboratories: ERGASIL condensation silicone; ERGAMIX and XILGUM addition silicones.

GUARANTEE

LASCOD SpA guarantees the quality of its products and undertakes to replace them if production defects are discovered within the expiry date. LASCOD SpA is not liable and does not admit any claims for any direct, indirect or accidental damages or damages caused by improper use of the material. Check the product's suitability prior to use.

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